

Technique for Making Provisional Restorations Quickly and Easily with Sil Tech Matrix (made from the lab's diagnostic wax-up)

1. After taking impression, etch one small spot (2 mm diameter circle) at the center of each tooth.
2. Paint bonding liquid on each spot.
3. Fill matrix carefully with temporary material (i.e. Exactatemp or Integrity), being careful not to trap air bubbles at the incisal edge.
4. Seat the Matrix, only pressing where there's an occlusal stop, so as not to overseat the matrix.
5. Mark the midline of the matrix to help orient it.
6. Hold for 2.5 minutes and gently peel matrix away from the gingival and facial surfaces, leaving the temporaries on the teeth.
7. Use a sickle scaler to remove thin flaching of temporary material.
8. Apply tints if you want more realism and glaze to make shiney.

